

# Treatment Protocol Hesturin Wrap for horses

PROBLEM	MODE	TIME	TREATMENT	POST-TREATMENT
Fresh wounds	P1	5 min	2x pr day, 3 days	1x pr day until healed
Fungus	P2	15 min	2x pr day, 7 days	1x pr day long as needed
Skin inflammation	P4	10 min	2x pr day, 5 days	1x pr day long as needed
Eczema	P1	15 min	1x pr day, 7 days	1x pr day long as needed
Cronic wounds	P3	10 min	2x pr day, 7 days	1x pr day until healed
Swelling	P5	10 min	2x pr day, 7 days	1x pr day long as needed
Tendon	P5	15 min	2x pr day, 10 days	1x pr day long as needed
Restitution	P2	20 min	1x after training	
Warm up	P6	10 min	1x before training	
Stress Relief	P4	10 min	1x pr days, 5 days	1x pr day long as needed
Circulation	P5	10 min	2x pr days, 5 days	1x pr day long as needed
Wellness	P3	20 min	1x pr days, 3 days	1x pr day long as needed
Health & Balance	P2	5 min	2x pr days, 3 days	1x pr day long as needed
Skin & Fur	P1	10 min	1x pr days, 7 days	1x pr day long as needed
Pain Relief	P6	15 min	2x pr days, 5 days	1x pr day long as needed
Cronic Conditions	P5	20 min	1x pr days	1x pr day long as needed

The condition treatment protocol serves as a guideline, indicating how you can use your IC Light Hesturin Wrap to treat your horse. The IC Light Hesturin Wrap is not intended as a replacement for standard medical procedures, medication and/or treatment. If you have any question, please contact your reseller or Caball Therapy Systems. Mail: [post@photizo.info](mailto:post@photizo.info) or [www.caball.biz](http://www.caball.biz)

